




1. Press « **Start** » button to switch on the device

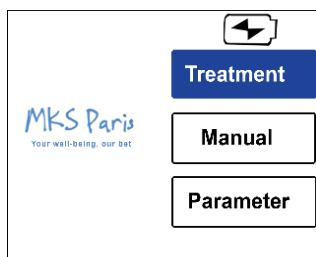


2. Select « **Treatment** » mode

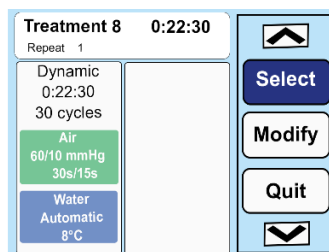

3. Select the desired treatment using the arrow buttons / , then press « **Select** »

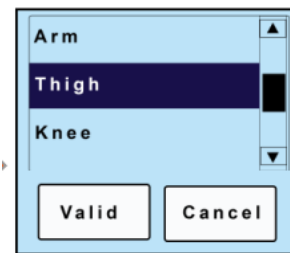


4. « **Validate** » the type of splint used for your cryotherapy treatment

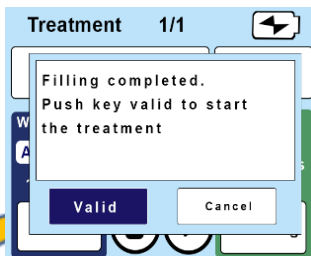


7. Please be patient during the splint filling. If the splint has yet been filled of water, press « **Validate** »




6. Please verify connexions between the device, the hose and the splint, then « **Validate** »

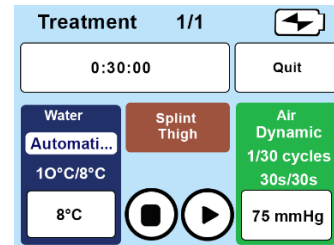

5. The selected treatment appears with its parameters

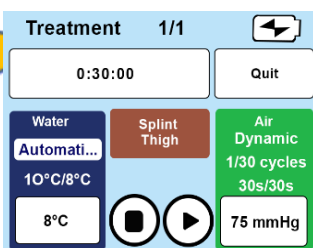

8. The cryotherapy treatment automatically starts after the splint filling

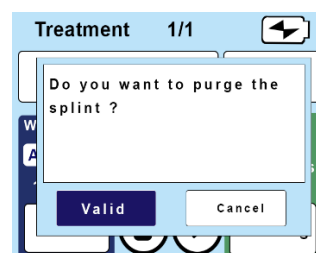

9. An audible « Beep » indicates the end of treatment, « **Validate** » to fill out the splint